

Product Description:

Green Stick is an impression material made of thermo plastic resin, in the form of green colored sticks, for recording negative replica of soft oral tissue and border moulding.

Indications:

1. Impression of oral tissue
2. Border moulding work

Contra Indications:

Not to be used on cut or wound surfaces or severe undercut areas.

Precautions:

Handle the packaging with care to avoid damage to the material within.

Procedure:

1. Green Stick when heated on spirit lamp or Bunsen burner softens.
2. Soften mass is spread on border of shellac trays which are then loaded in patient's mouth.
3. With temperature reducing mass would harden recording of the tissue.
4. Harden mass along with the tray is removed from patient's mouth.
5. The total exposure to intra oral tissues is 1 to 3 minutes.

Storage & Shelf Life:

Store away from sunlight in a cool and dry place. Handle the packaging with care to avoid damage to the material. Do not use after expiration date.

Disposal Conditions:

To be disposed of based on the prevailing rules of the country in which it is used.

Presentation:

Green Stick: 10 sticks – Approximately 90g

Note: The material has been developed for professional dental use and it is to be used by a dental professional only. Application should be carried out strictly according to the instruction for use. User is responsible for testing the material for its suitability. The manufacturer cannot be held liable for damages resulting from failure to observe the indication and procedure of use as mentioned in the instructions for use. The user should not use material for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.